

GENU REHA

Mobile knee brace

Ref: SRT 344



INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests.

These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

Available sizes

Sizes	1	2	3
Knee circumference (cm)	25-33	34-43	44-51
Reference	SRT344M1	SRT344M2	SRT344M3

The circumference of the knee is measured at the middle of the knee cap

Indications

It is used to stabilize the knee, in case of ligament instability, genu recurvatum.

Description

The orthosis is made of a metal frame, provided with:

- 2 hook and loop rigid straps on the front
- rear band that allows the adjustment of the hyperextension blocking angle.

Composition

The metal frame is made of 3 mm duralumin.

The 3 rigid straps for fixation are made of rigid fabric, provided with protection pads.

Positioning

The orthosis is placed on the knee making sure that the posterior band is positioned correctly, in the middle of the popliteal region. Fasten the 2 rigid straps and adjust the hyperextension locking angle on the posterior band.

Cleaning



To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage

At room temperature, in a dry space with good ventilation.

The legal guarantee of conformity according to Law no. 449/2003: **2 years.**

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: suportclienti@triamed.ro

WARRANTY COUPON 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller signature and stamp



Manufacturer: TRIAMED SRL

România/Timiș/Timișoara, Str. Calea Buziașului nr.77, Hala 1-C1/A

Tel/Fax: 00 40 256 487388

office@triamed.ro, triamed@gmail.ro

www.triamed.ro

Edition 1, Revision 1/09.11.2022