

PROCONFORT

Back support

Ref: SRT 102



INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

Available sizes

Sizes	1	2	3	4	5	6
Waist circumference (cm)	65-75	75-85	85-95	95-105	105-115	115-125
Reference	SRT102M1	SRT102M2	SRT102M3	SRT102M4	SRT102M5	SRT102M6

The circumference of the waist is measured

Indications

It is used in case of incipient intervertebral disc herniation, moderate low back pain, postoperative, in case of osteoporosis, for moderate stabilization of the lumbar spine.

Description

The support is made of elastic fabric, provided with:

- 4 integrated plastic rods for support, on the lumbar region
- 2 integrated plastic rods for support, on the abdominal region
- 2 elastic straps of 6 cm surround the waist, providing stability and compression
- a semi-rigid pad made of polyurethane foam, with textile liner, which produces a micromassage effect at lumbar level and helps to preserve local heat and protect the area. The shape and pressure points of the pad achieve a massage effect in the lumbar area. The lumbar pad is removable and can be positioned as needed.

Hook and loop fastener on the front side.

Width: front 16 cm, back 27,5 cm.

For an easy application it is provided with helping strips.

The 6 plastic rods stabilize the lumbar region, slightly reducing lordosis, and the compression exerted by the elastic bands stimulates local metabolism, reducing muscle tension and pain.

It is a traditional elastic corset that, due to the material, allows normal thoraco abdominal breathing, while ensuring uniform compression on the entire covered surface.

Composition

Elastic fabric: 50% polyester, 33% polyamide, 17% elastane, with a structure that allows high air permeability in both directions.

Positioning

The lumbar support is applied from back to front, the wider part is fixed on the lumbar region of the back, the narrow part is closed in the front, on the abdomen.

The double elastic strap of 6 cm surrounds the waist. The lumbar part is perfectly stretched, without creases, at a correct fixation.



Cleaning

To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage At room temperature, in a dry space with good ventilation.

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: suportclienti@triamed.ro

WARRANTY COUPON 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller name and signature



Manufacturer SC TRIAMED SRL

Production Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

Registered office Uliuc 262C | Sacoșu Turcesc | Timiș | România

Phone 00 40 256 48 73 88

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 3, Revision 3/21.08.2024