

## OMBIFIX SPORT

Abdominal support for umbilical hernia

Ref. SRT 149



### INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

#### Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here.

The product is intended for single patient use.

No liability is accepted for incorrect product use.

Guarantee is bound to statutory regulations.

#### Available sizes

Universal size

For waist circumference between 65 – 125 cm.

#### Indications

It is used in case of abdominal disorders and umbilical hernia.

#### Description

Support with anatomical design, made of elastic fabric, provided with one semirigid, standard dimension (polyethylene foam, Ø 11 cm) removable pad, on the front, at the level of the navel.

Hook and loop straps.

Width: 14 cm.

It is a traditional elastic corset that, due to the material, allows normal thoraco abdominal breathing, while ensuring uniform compression on the entire covered surface. The semirigid pad is detachable, it can be repositioned or removed. It provides protection and compression to the umbilical region. The support can be used in case of umbilical hernia, both before and after surgery.

#### Composition

Elastic fabric: PES Polyester – 40%, PA Polyamide – 40%, EA Elastan - 20%, with a structure that allows high air permeability in both directions.

Abdominal pad: PE Polyethylene foam – 100%, lined with soft textile.

#### Positioning

The orthosis is applied from the side, so that the pad is positioned at the level of the navel. It is closed with hook and loop fastener.

#### Cleaning



To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

#### Storage

At room temperature, in a dry space with good ventilation.

#### Commercial warranty of the product: 30 days.

**Date of production:** see packaging label.

**Validity date:** no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

**Deadline for communication of the solution: 15 days.**

Please send notifications and complaints to the following email address: [suportclienti@triamed.ro](mailto:suportclienti@triamed.ro)

<b>WARRANTY COUPON</b>	Batch and date of manufacture
30 days from the date of purchase	
Product reference	Date of purchase
The reason of complaint	Seller name and signature



**Manufacturer**

**SC TRIAMED SRL**

**Production** Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

**Registered office** Uliuc 262C | Sacoșu Turcesc | Timiș | România

**Phone** 00 40 256 48 73 88

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 1, Revision 1/05.03.2025