

TLS 01 JUNIOR

Thoraco lumbo sacral support

Ref: SRT 128

INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

Available sizes

Sizes	1	2	3
Waist circumference (cm)	45-52	52-59	59-65
Height of support on the back (cm)	35	40	45
Reference	SRT128M1	SRT128M2	SRT128M3

The circumference of the waist is measured

Indications

It is used after surgery or to support the thoracolumbar spine, in case of kyphosis and scoliosis.

Description

The support is made of elastic fabric, provided with:

- 2 lateral plastic splints for support
- 2 long, paravertebral plastic splints for support
- 2 plastic splints for the support of the abdominal area
- 2 elastic straps that offer an increased compression on the lumbar region
- 2 elastic straps that surround the shoulders and correct the position of the trunk. The straps have an elastic and a rigid component, are positioned over the head of the humerus, they are adjustable by cutting and are provided with a posterior guide to prevent their slipping into the armpit. The straps are closed at the front, on the corset. To shorten the straps, remove the "crocodile" fastener from the ends, cut the strap to the optimal length and reposition the fastener.

Hook and loop fastener on the front side.

The height of the support is between 35 – 45 cm, depending on size.

The 2 lateral and the 2 paravertebral plastic rods stabilize the thoracolumbar region, slightly reducing lordosis and kyphosis, and the compression exerted by the elastic bands stimulates local metabolism, reducing muscle tension and pain.

The 2 elastic bands that surround the shoulders and are fixed on the chest, correct by traction the position of the trunk. It is a traditional elastic corset that, due to the material, allows normal thoraco abdominal breathing, while ensuring uniform compression on the entire covered surface.

Purpose

The support stabilizes the thoracolumbar region, slightly reducing lordosis and kyphosis. The compression exerted by the elastic bands stimulates local metabolism, reducing muscle tension and pain. It corrects the position of the trunk.

Composition Textile fabrics, with a structure that allows high air permeability in both directions:

- PES Polyester - 50%, PA Polyamide - 33%, EA Elastan - 17%
- PES Polyester - 75%, CO Cotton - 25%
- PU Polyurethane - 36%, PA Polyamide - 30%, PES Polyester - 24,5%, EA Elastan - 9,5%

Positioning

The orthosis is placed from rear to front, the wide part is fixed on the lumbar region of the back, the narrow part will be fastened in the front, on the abdomen. The 2 long bands are passed around the shoulders, crossed in the back and are fixed in the front, on the thorax. The lumbar part is perfectly stretched, without creases, at a correct application.



Cleaning

To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage At room temperature, in a dry space with good ventilation.



Manufacturer SC TRIAMED SRL

Production Calea Buziașului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

Registered office Str. Cosminului.nr.12 | Timișoara | Timiș | România

Phone 00 40 256 487388

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 6, Revision 0/20.04.2025



Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: suportclienti@triamed.ro

WARRANTY COUPON	Batch and date of manufacture
30 days from the date of purchase	
Product reference	Date of purchase
The reason of complaint	Seller name and signature

SR EN ISO 9001:2015

SR EN ISO 13485:2016



Manufacturer SC TRIAMED SRL

Production Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

Registered office Str. Cosminului.nr.12 | Timișoara | Timiș | România

Phone 00 40 256 487388

office@triamed.ro | triamed@gmail.com | www.triamed.ro