


THORASPORT

Rigid thoracic support

Ref. SRT 147

INDICATIONS

It is used to correct vicious positions of the thoracic spine, or in case of broken collarbone.

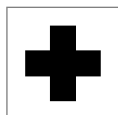

MEDICAL



SPORT



DAILY

DESCRIPTION

Support with anatomical design, made of soft fabric that does not harm while wearing provided with:


- one posterior component that supports the thoracic spine, made of rigid fabric (does not stretch)
- 2 plastic paravertebral splints, with the length of 18 cm, for support
- 2 bands that surround the shoulders, correcting by traction the position of the trunk.

The bands have an elastic and a rigid component, they are positioned over the head of the humerus and are adjustable by cutting.


In order to shorten the bands, remove the "crocodile" type hook and loop fastener from the edges, cut the band to optimal length and reposition the hook and loop. Hook and loop fastener on the front side.

COMPOSITION

The orthosis is made of two-layer and three-layer laminated textile.

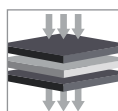

AIR FLOW

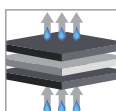

WATER
TRANSFER

AVAILABLE COLOURS

Black



AVAILABLE SIZES

Universal size

For thorax circumference between 75 – 135 cm.

