

## TRIAGEN

Mobile knee brace with lateral articulations

Ref: SRT 319

### INSTRUCTIONS FOR USE



Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

#### Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

#### Available sizes

Sizes	1	2	3	4	5	6	7	8
Knee circumference (cm)	33-35	35-37	37-39	39-41	41-43	43-45	45-47	47-49
Reference	SRT319M1	SRT319M2	SRT319M3	SRT319M4	SRT319M5	SRT319M6	SRT319M7	SRT319M8

The circumference of the knee is measured at the middle of the knee cap

#### Indications

It is used in lateral instability of the knee, traumas or injuries at the level of the knee, inflammatory processes, for postoperative recovery.

#### Description

The orthosis is made of textile, open in the front, provided with:

- a non-slip silicone elastic band (10mm) inside the upper part
- 2 metal splints with mobile articulations, one on each side
- 2 straps provided with elastic hook and loop fastener
- cutout in the popliteal region for comfort.

Hook and loop fastening on the front side.

Length: 32 cm.

The orthosis is a support for the knee joint, the 2 polycentric, lateral metal splints providing stability and safety in use. Due to its particular shape, it can be used also in case of a deformed knee. The compression can be adjusted by the frontal closure system and the 2 fastening straps.

#### Composition

Three-layer laminated textile. The laminated textile ensemble behaves like a moisture regulator, ensuring a quick transfer of the moisture to outward.

#### Positioning

The orthosis is placed from back to front, making sure that the knee cap is positioned correctly, in the middle of the orthosis and it remains free. The 2 straps with elastic hook and loop are fixed to the optimal compression level.



#### Cleaning

To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

**Storage** At room temperature, in a dry space with good ventilation.

**Commercial warranty of the product: 30 days.**

**Date of production:** see packaging label.

**Validity date:** no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

**Deadline for communication of the solution: 15 days.**

Please send notifications and complaints to the following email address: [suportclienti@triamed.ro](mailto:suportclienti@triamed.ro)

<b>WARRANTY COUPON</b>	Batch and date of manufacture
30 days from the date of purchase	
Product reference	Date of purchase
The reason of complaint	Seller name and signature



**Manufacturer SC TRIAMED SRL**

**Production** Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

**Registered office** Uliuc 262C | Sacoșu Turcesc | Timiș | România

**Phone** 00 40 256 48 73 88

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 3, Revision 3/13.08.2024