

## AKTIV

Mobile knee brace with patellar support and lateral articulations

Ref: SRT 304

## INSTRUCTIONS FOR USE



Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

### Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

### Available sizes

Sizes	1	2	3	4	5	6	7	8
Knee circumference (cm)	33-35	35-37	37-39	39-41	41-43	43-45	45-47	47-49
Reference	SRT304M1	SRT304M2	SRT304M3	SRT304M4	SRT304M5	SRT304M6	SRT304M7	SRT304M8

The circumference of the knee is measured at the middle of the knee cap

### Indications

It is used in lateral instability of the knee, traumas or injuries at the level of the knee, inflammatory processes, for postoperative recovery.

### Description

It is a tubular, closed orthosis, made of textile, provided with:

- a non-slip silicone elastic band (10mm) inside the upper part
- 2 metal splints with mobile articulations, one on each side
- 2 straps of elastic fastening material. The elastic fastening follows the conical shape of the thigh, preventing the orthosis from slipping
- patellar support (polyethylene foam).

Length: 32 cm.

The orthosis is a support for the knee joint, the 2 polycentric, lateral metal splints providing stability and safety in use.

Due to its tubular, sleeve shape, it is not recommended to be used in case of oedema or significant swelling.

### Composition

Neoprene coated on both sides with fabric: PES Polyester - 100%, SBR Styren-butadien Rubber – 100%, PA Nylon – 100%.

### Positioning

The orthosis is placed on the leg so that the patella support surrounds the knee cap and the lateral splints are parallel with the median axis of the leg and their articulations must correspond to the articulation of the knee. The 2 straps are passed through buckles and are fixed to the optimal compression.

### Cleaning



To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

**Storage** At room temperature, in a dry space with good ventilation.

### Commercial warranty of the product: 30 days.

**Date of production:** see packaging label.

**Validity date:** no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

**Deadline for communication of the solution: 15 days.**

Please send notifications and complaints to the following email address: [suportclienti@triamed.ro](mailto:suportclienti@triamed.ro)

<b>WARRANTY COUPON</b>	Batch and date of manufacture
30 days from the date of purchase	
Product reference	Date of purchase
The reason of complaint	Seller name and signature



**Manufacturer SC TRIAMED SRL**

**Production** Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

**Registered office** Uliciuc 262C | Sacoșu Turcesc | Timiș | România

**Phone** 00 40 256 48 73 88

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 3, Revision 4/31.01.2025