

GENU REHA COOL ROM 02

Mobile knee orthosis, telescopic, with adjustable articulations

Ref. SRT 359

INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

Available sizes

Universal size, for a knee with the circumference between 30 – 52 cm.

The orthosis can be used on both the left and the right knee.

Indications

It is used for postoperative immobilization, the control of the flexion and extension of the knee joint, traumas or injuries at the level of the knee or minor fractures.

Description

The orthosis is made of 2 metal splints with adjustable length, with mobile, adjustable and lockable articulations, one on each side.

The orthosis is provided with 4 textile pads for calf protection and 4 textile pads for thigh protection.

The adjustment device is easily accessed, the angle is adjusted with 2 clips. These clips are operated by pressing and moving them to the desired position.

The length is adjusted between 50 - 63 cm, using buttons located on the metal splint rods.

The possibilities of limiting the movement are:

- **flexion** 0°, 15°, 30°, 45°, 60°, 75°, 90°, 105°, 120°

- **extension** 0°, 15°, 30°, 45°, 60°, 75°, 90°

- 2 hook and loop straps on the leg
- 2 hook and loop straps on the thigh.

The length of the 4 straps is adjustable by cutting and repositioning the hook and loop.

Length: 50 – 63 cm.

Due to its particular shape, it can be used also in case of a deformed knee. The compression can be adjusted by the frontal closure system and the 4 fastening straps.

Purpose

The orthosis is a support for the knee joint, the 2 lockable, lateral metal splints providing stability and safety in use.

Composition

Laminated textile. The laminated textile ensemble behaves like a moisture regulator, ensuring a quick transfer of the moisture to outward.

Positioning

The orthosis is placed on the knee from the back to the front, making sure that the 2 metal splints are positioned near the joint space of the knee and parallel in antero-posterior plane. The metal splints are adjusted only according to the recommendations of the specialized physician. The 4 straps provided with "crocodile type" fastening, are fixed to the optimal compression.

If it is necessary to adjust the position of the orthosis, the adjustment is made by fully opening the hook and loop fastener and repositioning the orthosis higher or lower on the body segment, depending on the need. **To avoid the damage of the material, do not pull the orthosis while is fixed.**

Cleaning



To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage

At room temperature, in a dry space with good ventilation.

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.


Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: suportclienti@triamed.ro

WARRANTY COUPON 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller name and signature

SR EN ISO 9001:2015

SR EN ISO 13485:2016

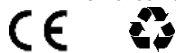

Manufacturer SC TRIAMED SRL

Production Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

Registered office Str. Cosminului.nr.12 | Timișoara | Timiș | România

Phone 00 40 256 487388

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 6, Revision 0/31.07.2025