



TRIAMED

Shoulder elbow wrist hand brace

Ref. SRT 219

INSTRUCTIONS FOR USE



Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here.

The product is intended for single patient use.

No liability is accepted for incorrect product use.

Guarantee is bound to statutory regulations.

Available sizes

Universal size

For arm and thorax circumference between 90 – 150 cm.

Indications

It is used in case of humerus sprain, humerus collar fracture, the humeral tuberosity fracture without movement, shoulder blade fracture, scapulohumeral dislocation.

Description

Orthosis with anatomical design, made of textile, provided with 2 components:

- one arm support with 2 straps adjustable by cutting them and repositioning the “crocodile” type hook and loop fastening
- one adjustable thoracic belt to be fixed around the thorax and arm after the immobilization of the arm. The regulation is done by cutting at the level of the stitches to avoid fabric breaking.

Hook and loop fastener.

The orthosis is a support for the shoulder, arm and forearm. Due to the thoracic belt the support does not allow the abduction of the arm and external rotation.

Composition

Multi-layers laminated textile: PA Polyamide – 68%, PU Polyurethane – 32%. The laminated textile ensemble behaves like a moisture regulator, ensuring a quick transfer of the moisture to outward.

Positioning

The arm is positioned in the support at a 90° angle and the bands are fastened at the desired height by cutting and repositioning the “crocodile” type hook and loop fastening. After this, the belt is fastened around the chest and arm and closed on the front so that the arm is immobilized. The regulation is done by cutting at the level of the stitches to avoid fabric breaking.



Cleaning

To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage

At room temperature, in a dry space with good ventilation.

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: suportclienti@triamed.ro

WARRANTY COUPON	Batch and date of manufacture
30 days from the date of purchase	
Product reference	Date of purchase
The reason of complaint	Seller name and signature



Manufacturer

SC TRIAMED SRL

Production Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

Registered office Uliuc 262C | Sacoșu Turcesc | Timiș | România

Phone 00 40 256 48 73 88

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 3, Revision 4/05.03.2025