

GEROMED 02

Mobile knee brace

Ref: SRT 340

INSTRUCTIONS FOR USE



Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

Available sizes

Sizes	1	2	3	4	5
Knee circumference (cm)	33-36	36-39	39-42	42-45	45-49
Reference	SRT340M1	SRT340M2	SRT340M3	SRT340M4	SRT340M5

The circumference of the knee is measured at the middle of the knee cap

Indications

It is used in cricks, luxations, post consolidated fractures, inflammatory processes, lateral knee joint instability. It is recommended both as orthopedic treatment and postoperative.

Description

The orthosis is open at the back, made of textile, provided with:

- a non-slip silicone elastic band (10mm) inside the upper part
- 4 spring type metal splints, 2 on each side
- patellar support (polyethylene foam)
- 3 neoprene fastening straps: 2 front and one rear closure
- 2 straps of elastic fastening material. The elastic fastening follows the conical shape of the thigh, preventing the orthosis from slipping.

Length: 32 cm.

The orthosis is a support for the knee joint, the 4 spring type metal splints, 2 for each side providing stability and safety in use. Due to its particular shape, it can be used also in case of a deformed knee. The compression can be adjusted by the back closure system.


Composition

Neoprene coated on both sides with fabric: PES Polyester - 100%, SBR Styren-butadien Rubber – 100%, PA Nylon – 100%.

Positioning

The orthosis is placed from front to back, making sure that the knee cap is positioned correctly, in the middle of the orthosis and that it remains free. The straps are fixed to the optimal compression.

Cleaning

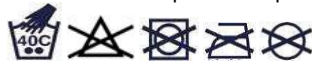

To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage At room temperature, in a dry space with good ventilation.

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: suportclienti@triamed.ro

WARRANTY COUPON	Batch and date of manufacture
30 days from the date of purchase	
Product reference	Date of purchase
The reason of complaint	Seller name and signature



Manufacturer SC TRIAMED SRL

Production Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

Registered office Uliuc 262C | Sacoșu Turcesc | Timiș | România

Phone 00 40 256 48 73 88

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 3, Revision 4/05.03.2025