

## TRIACROSS

Mobile knee brace

Ref: SRT 317

### INSTRUCTIONS FOR USE



Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

#### Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

#### Available sizes

Sizes	1	2	3	4	5	6	7	8
Knee circumference (cm)	33-35	35-37	37-39	39-41	41-43	43-45	45-47	47-49
Reference	SRT317M1	SRT317M2	SRT317M3	SRT317M4	SRT317M5	SRT317M6	SRT317M7	SRT317M8

The circumference of the knee is measured at the middle of the knee cap

#### Indications

It is used in cricks, luxations, post consolidated fractures, lateral knee joint instability, inflammatory processes, anterior cruciate ligament injury. It is recommended both as orthopedic treatment and postoperative.

#### Description

It is a tubular, closed orthosis, made of textile, provided with:

- a non-slip silicone elastic band (10mm) inside the upper part
- 4 spring type metal splints, 2 on each side
- patellar support (polyethylene foam)
- elastic strap crossed in the front, under the patellar support, and on the back, in the popliteal region.

Length: 29 cm.

The orthosis is a support for the knee joint, the 2 spring type metal splints for each side providing stability and safety in use.

Due to its tubular, sleeve shape, it is not recommended to be used in case of oedema or significant swelling.

#### Composition

Neoprene coated on both sides with fabric: PES Polyester - 100%, SBR Styren-butadien Rubber – 100%, PA Nylon – 100%.

#### Positioning

The orthosis is placed on the leg so that the patella support surrounds the knee cap and the lateral splints are parallel with the median axis of the leg.

The elastic strap is crossed in the front, under the patellar support, on the back, in the popliteal region and it is closed in the front, on the thigh, above the patellar support, at the optimal compression.

#### Cleaning



To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

**Storage** At room temperature, in a dry space with good ventilation.

#### Commercial warranty of the product: 30 days.

**Date of production:** see packaging label.

**Validity date:** no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

#### Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: [suportclienti@triamed.ro](mailto:suportclienti@triamed.ro)

<b>WARRANTY COUPON</b> 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller name and signature



**Manufacturer SC TRIAMED SRL**

**Production** Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

**Registered office** Uliuc 262C | Sacoșu Turcesc | Timiș | România

**Phone** 00 40 256 48 73 88

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 3, Revision 4/05.03.2025