

GENU REHA PO

Mobile knee brace with adjustable articulations

Ref: SRT 309

INSTRUCTIONS FOR USE



Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

Available sizes

Sizes	1	2	3	4	5	6	7	8
Knee circumference (cm)	33-35	35-37	37-39	39-41	41-43	43-45	45-47	47-49
Reference	SRT309M1	SRT309M2	SRT309M3	SRT309M4	SRT309M5	SRT309M6	SRT309M7	SRT309M8

The circumference of the knee is measured at the middle of the knee cap

Indications

It is used for postoperative recovery, control of the flexion and extension of the knee joint.

Description

The orthosis is open in the front, made of textile, provided with:

- a non-slip silicone elastic band (10mm) inside the upper part
- 2 metal splints with mobile, adjustable and lockable articulations, one on each side. The adjustment device is easily accessed on the sides, covered with a plastic cap. The angle is adjusted with 2 rivets. The possibilities of limiting the movement are:
 - flexion 10°, 20°, 30°, 60°, 90°
 - extension 10°, 20°, 30°, 60°, 90°
- 2 hook and loop straps on the leg
- 2 hook and loop straps on the thigh
- cutout in the popliteal region for comfort.

Length: 44 cm.

The orthosis is a support for the knee joint, the 2 lateral metal splints providing stability and safety in use. Due to its particular shape, it can be used also in case of a deformed knee. The compression can be adjusted by the frontal closure system and the 4 fastening straps.

Composition

Three-layer laminated textile. The laminated textile ensemble behaves like a moisture regulator, ensuring a quick transfer of the moisture to outward.

Positioning

The orthosis is placed on the knee from the back to the front, making sure that the knee cap is positioned correctly, in the middle of the orthosis and that it remains free. The 4 straps are fixed to the optimal compression.

Cleaning



To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage At room temperature, in a dry space with good ventilation.

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: suportclienti@triamed.ro

WARRANTY COUPON 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller name and signature



Manufacturer SC TRIAMED SRL

Production Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

Registered office Uliuc 262C | Sacoșu Turcesc | Timiș | România

Phone 00 40 256 48 73 88

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 3, Revision 5/19.07.2024