

**SENIOR 01**  
Back support  
Ref: SRT 120



## INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests.

These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

The orthosis is used according to the recommendations of the specialized physician.

**Choosing the size** - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

### Available sizes

Sizes	1	2	3	4	5	6
Waist circumference (cm)	65-75	75-85	85-95	95-105	105-115	115-125
Reference	SRT120M1	SRT120M2	SRT120M3	SRT120M4	SRT120M5	SRT120M6

The circumference of the waist is measured

### Indications

It is used in case of incipient intervertebral disc herniation, moderate low back pain, postoperative, in case of osteoporosis, for moderate stabilization of the lumbar spine.

### Description

The support is made of textile, provided with 4 integrated plastic rods for support.

Hook and loop fastener on the front side.

Width: front 16 cm, back 27 cm.

For an easy application it is provided with helping strips.

The 4 plastic rods stabilize the lumbar region, slightly reducing lordosis, and the compression exerted by the elastic bands stimulates local metabolism, reducing muscle tension and pain.

### Composition

Three-layer laminated textile. The laminated textile ensemble behaves like a moisture regulator, ensuring a quick transfer of the moisture to outward.

### Positioning

The lumbar support is applied from back to front, the wider part is fixed on the lumbar region of the back, the narrow part is closed in the front, on the abdomen. The lumbar part is perfectly stretched, without creases, at a correct fixation.

### Cleaning



To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

### Storage

At room temperature, in a dry space with good ventilation.

**The legal guarantee of conformity** according to Law no. 449/2003: **2 years.**

**Commercial warranty of the product: 30 days.**

**Date of production:** see packaging label.

**Validity date:** no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

**Deadline for communication of the solution: 15 days.**

Please send notifications and complaints to the following email address: [supportclienti@triamed.ro](mailto:supportclienti@triamed.ro)

<b>WARRANTY COUPON</b> 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller signature and stamp



**Manufacturer: TRIAMED SRL**

România/Timiș/Timișoara, Str. Calea Buziașului nr.77, Hala 1-C1/A

Tel/Fax: 00 40 256 487388

[office@triamed.ro](mailto:office@triamed.ro), [triamed@gmail.com](mailto:triamed@gmail.com)

[www.triamed.ro](http://www.triamed.ro)

Edition 3, Revision 2/07.11.2022