

ANKLEFIX ACTIV

Mobile ankle support with plastic side splints

Ref. SRT 314



INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here.

The product is intended for single patient use.

No liability is accepted for incorrect product use.

Guarantee is bound to statutory regulations.

Available sizes

Universal size, for left and right versions

For ankle with circumference between 20 - 30 cm.

Indications

It is used in cricks, luxations, post consolidated fractures, inflammatory processes, after the removal of the orthopedic plaster cast.

Description

Orthosis with anatomical design, made of 2 rigid, plastic side splints, with protection pads made of polyethylene foam, covered with soft textile and is provided with:

- 2 lateral fastening straps provided with hook and loop
- one fastening strap at heel level that allows the adjustment of the width.

Height: 25 cm.

The orthosis is a support for the ankle, the 2 rigid, plastic side splints ensure stability and safety in use.

Composition


Side splints: PP Polypropylene – 100%.

Protection pads: PA Polyamide – 100%. Are manufactured by thermoforming and have a shape that follows the anatomical shape of the ankle.

Positioning

The orthosis is placed under the heel, after which the rigid parts of the orthosis are lifted so that the ankle is framed on the lateral sides. It is fixed with the hook and loop straps to the optimal compression.

Cleaning

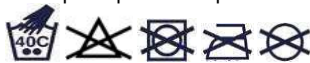

To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage

At room temperature, in a dry space with good ventilation.

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: suportclienti@triamed.ro

WARRANTY COUPON 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller name and signature



Manufacturer

SC TRIAMED SRL

Production Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

Registered office Uliuc 262C | Sacoșu Turcesc | Timiș | România

Phone 00 40 256 48 73 88

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 3, Revision 4/03.03.2025