

AEROTEC LS PLUS

Rigid back support

Ref: SRT 130



INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests.

These orthoses meet the essential requirements of Government's Decision 54/2009 and have registration Certificates from the Ministry of Health.

These products satisfy the European Medical Device Directive 93/42/CEE (RD.1591/2009).

The orthosis is used according to the recommendations of the specialized physician. The product must be adjusted by the orthopaedic technician!

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

Available sizes

Sizes	1	2	3	4	5	6
Waist circumference (cm)	65-75	75-85	85-95	95-105	105-115	115-125
Reference	SRT130M1	SRT130M2	SRT130M3	SRT130M4	SRT130M5	SRT130M6

The circumference of the waist is measured

Indications

It is used in case of debutant intervertebral disc herniation, moderate low back pain, postoperative, in case of osteoporosis, for moderate stabilization of the lumbar spine.

Description

The support is made of breathable, rigid textile, provided with:

- 6 integrated plastic rods for support, on the lumbar region
- 2 integrated plastic rods for support, on the abdominal region (positioned inclined to allow flexion of the thigh on the abdomen)
- 2 elastic straps of 5,5 cm around the waist, that provide compression.

Hook and loop fastener on the front side.

Width: front 18 cm, back 30 cm.

For an easy application, it is provided with helping strips.

The 6 plastic rods stabilize the lumbar region, slightly reducing lordosis, and the compression exerted by the elastic bands stimulates local metabolism, reducing muscle tension and pain.

Composition

The orthoses are made of three-layer laminated textile:

- **Spaziato** – spatialized 3D mesh with a specific shape, similar to the honeycomb, with polyester thread. With this particular look allows high air permeability in both directions.

• **Moltopren (MTP)** - air-permeable open-cell polyurethane foam (airflow in both directions is inversely proportional to foam density) with high moisture absorbing properties.

• **Jersey cotone SOFT** - 100% cotton or cotton with spandex - elastane (for increased elasticity).

SOFT

The code for the balsamic treatment of the cotton fabric to provide softness to the touch and to avoid skin irritation.

Positioning

The lumbar support shall be applied from back to front, the wider part is fixed on the lumbar region of the back, the narrow part shall be closed in the front, on the abdomen. The two, 5,5 cm wide elastic straps are fixed around the waist. The lumbar part is perfectly stretched, without creases, at a correct fixation.

Cleaning



To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage

At room temperature, in a dry space with good ventilation.

The legal guarantee of conformity according to Law no. 449/2003: **2 years.**

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: suportclienti@triamed.ro

WARRANTY COUPON 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller signature and stamp



Manufacturer: TRIAMED SRL

România/Timiş/Timişoara, Str. Calea Buziaşului nr.77, Hala 1-C1/A

Tel/Fax: 00 40 256 487388, Tel: 00 40 765 240037

office@triamed.ro, triamed@gmail.ro

www.triamed.ro