

TRIA SAB

Arm abduction brace to 45°/60°

Ref: SRT 221

INSTRUCTIONS FOR USE



Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests.

These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

Available sizes

Sizes	1	2	3
Arm and thorax circumference (cm)	85-105	105-125	125-145
Arm and palm length (without fingers) (cm)	36	39	43
Reference	SRT221M1	SRT221M2	SRT221M3

The circumference of the arm and thorax is measured

Indications

It is used in plastical surgery, orthopedics, postoperative, post traumatic arm and shoulder injuries, scapulohumeral fractures.

Description

The orthosis has 3 components:

- polyurethane foam (sponge) cushion for arm supporting covered with fabric, designed with 2 angles: 45° and 60°. The abdominal cushion is fixed to the abdomen by a positioning belt provided with 2 double Lock buckles for adjustment
- forearm fixing component made of polyurethane foam (sponge)
- thoracic harness made of 4 straps adjustable by cutting them and repositioning the "crocodile" type hook and loop fastening, provided with 2 rings and 2 protective sleeves.

The position of the arm can be at 45° or 60° reported to the chest, by changing the position of the support cushion.

It can be used both on the left and on the right side.

Hook and loop fastener.

Composition

The orthosis is made of polyurethane foam (sponge) covered with fabric: 100% polyamide.

The textile material made of polyamide fiber has a high resistance to chemical and biological agents, resists abrasion and does not absorb water.

Positioning

The triangular sponge is fixed between the chest and the arm so that the abduction angle of the arm corresponds to the instructions of the specialist. The harness is fixed on the sternum, one long side of the harness is passed over the healthy shoulder and the other subaxillary and both are fastened on the back side of the triangular sponge. It is fixed to the optimum length by cutting the strips and repositioning the "crocodile" type hook and loop fastening. The short straps of the harness are fixed on the front of the triangular sponge. The forearm sponge is adapted to the triangle, so it can support the forearm and the hand.



Cleaning

To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage

At room temperature, in a dry space with good ventilation.

The legal guarantee of conformity according to Law no. 449/2003: **2 years.**

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: suportclienti@triamed.ro

WARRANTY COUPON 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller signature and stamp



Manufacturer: TRIAMED SRL

România/Timiș/Timișoara, Str. Calea Buziașului nr.77, Hala 1-C1/A

Tel/Fax: 00 40 256 487388

office@triamed.ro, triamed@gmail.com

www.triamed.ro

Edition 3, Revision 2/31.10.2022