

THORATEX 04

Rigid thoracic support

Ref. SRT 140

INSTRUCTIONS FOR USE



Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here.

The product is intended for single patient use.

No liability is accepted for incorrect product use.

Guarantee is bound to statutory regulations.

Available sizes

Universal size

For thorax circumference between 75 – 135 cm.

Indications

It is used to correct vicious positions of the thoracic spine, or in case of broken collarbone.

Description

Support with anatomical design, made of 3 rigid textile fixing straps:

- 2 straps that go around the shoulders correcting the body posture by traction
- one back strap, used for the final adjustment.

The 3 straps are adjustable. To shorten the straps, remove the "crocodile" type fastening from the extremities, cut the strap to the optimum length on the prefabricated transverse marks and reposition the hook and loop. The two straps that surround the shoulders are provided with sleeves for underarm protection.

The orthosis is made of a soft, rigid fabric that does not harm while wearing. The big surface of the hook and loop fastening allows a good adjustment and strong traction, helping the clavicle to return to its anatomical shape after fracture.

For a better result it is recommended to readjust during wearing.

Hook and loop fastener on the front side.

Composition

The rigid textile made of polyamide fiber has a high resistance to chemical and biological agents, resists abrasion and does not absorb water.

Positioning

Place the orthosis on the dorsal segment of the spine and adjust the bands around the shoulders to the optimum level of traction. The intensity of the traction can be adjusted with the help of the back band.



Cleaning

To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage

At room temperature, in a dry space with good ventilation.

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: suportclienti@triamed.ro

WARRANTY COUPON 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller name and signature



Manufacturer

SC TRIAMED SRL

Production Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

Registered office Uliuc 262C | Sacoșu Turcesc | Timiș | România

Phone 00 40 256 48 73 88

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 1, Revision 0/30.10.2023