

## CARPOFIX MANU

Fixed wrist hand brace

Ref: SRT 203

### INSTRUCTIONS FOR USE



Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests.

These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

The orthosis is used according to the recommendations of the specialized physician.

**Choosing the size** - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

#### Available sizes, left/right versions

Sizes	1	2	3	4	5
Wrist circumference (cm)	15-17	17-19	19-21	21-23	23-25
Reference	SRT203M1	SRT203M2	SRT203M3	SRT203M4	SRT203M5

After the code must be added **L** for Left, **R** for Right

The circumference of the wrist is measured

#### Indications

It is used in sprains, dislocations, arthrosis, carpal tunnel syndrome, radial nerve paralysis, tenosynovitis, fractures at the level of the area, both as orthopedic and postoperative treatment.

#### Description

Brace made of textile, provided with:

- one mouldable metal palm splint
- 2 inserted plastic splints on the dorsal side of the hand.

Hook and loop fastener with 3 straps provided with buckles, on the dorsal side of the hand.

Length: 24 cm.

The orthosis stabilizes and immobilizes the radiocarpal joint.

#### Composition

Three-layer laminated textile. The laminated textile ensemble behaves like a moisture regulator, ensuring a quick transfer of the moisture to outward.

#### Positioning

The orthosis is positioned so that the metal splint supports the palm. It is fastened on the back side of the forearm, passing the fastening bands through the 3 buckles and adjusting it to the optimal compression.

#### Cleaning



To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

#### Storage

At room temperature, in a dry space with good ventilation.

**The legal guarantee of conformity** according to Law no. 449/2003: **2 years.**

**Commercial warranty of the product: 30 days.**

**Date of production:** see packaging label.

**Validity date:** no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

**Deadline for communication of the solution: 15 days.**

Please send notifications and complaints to the following email address: [suportclienti@triamed.ro](mailto:suportclienti@triamed.ro)

<b>WARRANTY COUPON</b> 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller signature and stamp



**Manufacturer: TRIAMED SRL**

România/Timiş/Timişoara, Str. Calea Buziaşului nr.77, Hala 1-C1/A

Tel/Fax: 00 40 256 487388

[office@triamed.ro](mailto:office@triamed.ro), [triamed@gmail.ro](mailto:triamed@gmail.ro)

[www.triamed.ro](http://www.triamed.ro)

Edition 3, Revision 2/31.10.2022