

## TRIAFIX ELBOW

Fixed elbow wrist hand brace

Ref: SRT 214



### INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests.

These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

The orthosis is used according to the recommendations of the specialized physician.

**Choosing the size** - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

#### Available sizes, left/right versions

| Sizes                    | 1        | 2        | 3        | 4        | 5        | 6        |
|--------------------------|----------|----------|----------|----------|----------|----------|
| Elbow circumference (cm) | 22-24    | 24-26    | 26-28    | 28-30    | 30-32    | 32-34    |
| Forearm length (cm)      | 22-25    | 22-25    | 25-28    | 25-28    | 28-31    | 28-31    |
| Reference                | SRT214M1 | SRT214M2 | SRT214M3 | SRT214M4 | SRT214M5 | SRT214M6 |

After the code must be added **L** for Left, **R** for Right

The circumference of the elbow and the length of the forearm are measured

#### Indications

It is used in tendinopathy, carpal tunnel syndrome, radial nerve paralysis, cubital tunnel syndrome, fracture at the level of the wrist or elbow, both as orthopedic treatment and postoperative.

#### Description

Brace made of textile, provided with:

- one mouldable metal palm splint
- 2 fixed metal elbow splints set at 90°, mouldable, of the same length as the orthosis
- 3 straps with hook and loop (2 on the forearm and one on the arm)
- adjustable support strap (with "crocodile" type fastener and protection sleeve), optional. The positioning is done on the first fixing strap on the forearm and over the opposite shoulder.

Hook and loop fastener on the dorsal side of the hand.

Length: 51 - 57 cm.

The orthosis stabilizes and immobilizes the elbow and the radiocarpal joint.

#### Composition

Three-layer laminated textile. The laminated textile ensemble behaves like a moisture regulator, ensuring a quick transfer of the moisture to outward.

#### Positioning

The orthosis is positioned so that one metal splint supports the palm and the other 2 frame the elbow. The orthosis is fastened with the 3 straps to the optimal compression.

#### Cleaning



To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

#### Storage

At room temperature, in a dry space with good ventilation.

**The legal guarantee of conformity** according to Law no. 449/2003: **2 years.**

**Commercial warranty of the product: 30 days.**

**Date of production:** see packaging label.

**Validity date:** no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

**Deadline for communication of the solution: 15 days.**

Please send notifications and complaints to the following email address: [suportclienti@triamed.ro](mailto:suportclienti@triamed.ro)

|   |                               |
|---|-------------------------------|
| <b>WARRANTY COUPON</b><br>30 days from the date of purchase | Batch and date of manufacture |
| Product reference   | Date of purchase              |
| The reason of complaint                                     | Seller signature and stamp    |



**Manufacturer: TRIAMED SRL**

România/Timiș/Timișoara, Str. Calea Buziașului nr.77, Hala 1-C1/A

Tel/Fax: 00 40 256 487388

[office@triamed.ro](mailto:office@triamed.ro), [triamed@gmail.ro](mailto:triamed@gmail.ro)

[www.triamed.ro](http://www.triamed.ro)

Edition 3, Revision 2/31.10.2022