

HERNIA DUO

Hernia support
Ref. SRT151



INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here.

The product is intended for single patient use.

No liability is accepted for incorrect product use.

Guarantee is bound to statutory regulations.

Available sizes

Universal size

For waist circumference between 75 – 135 cm.

Indications

It is used in case of small or medium reducible inguinal hernia.

Description

Support with anatomical design, made of textile material, provided with:

- 2 leg straps
- 2 pressure pads, individually removable.

Hook and loop closure on the side. The hip textile band is provided on one end with "crocodile" type fastener, which allows the adjustment of the length by cutting it. The 2 leg straps are also adjustable by cutting, provided with "crocodile" type fastener.

The orthosis is small and can be easily worn under the clothes. The inguinal hernia suport has the advantage of individual positioning of each pad thanks to the hook and loop fastening and at the same time, it offers the possibility of removing one of the pads, thus becoming unilateral. The strap on the unused side can be cut off and the support can be used on the left or on the right.

The orthosis provides support and comfort in case of small or medium reducible inguinal hernia.

Composition

Elastic fabric: PES Polyester - 50%, PA Polyamide - 30%, EL Elastan - 20%, with a structure that allows high air permeability in both directions.

Two-layer laminated textile: PA Polyamide - 100%.

Polyethylene foam (compression pad).

Positioning

The orthosis is positioned, it is closed laterally with the straps provided with "crocodile" fasteners. The length of the straps can be adjusted in advance. The front fixing straps are passed behind and adjusted so that the pads are well fixed.



Cleaning

To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage

At room temperature, in a dry space with good ventilation.

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: suportclienti@triamed.ro

WARRANTY COUPON	Batch and date of manufacture
30 days from the date of purchase	
Product reference	Date of purchase
The reason of complaint	Seller name and signature



Manufacturer

SC TRIAMED SRL

Production Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

Registered office Uliuc 262C | Sacoșu Turcesc | Timiș | România

Phone 00 40 256 48 73 88

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 1, Revision 1/09.12.2024