

## ARMFIX STRONG

Shoulder orthosis  
Ref. SRT 238

### INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

#### Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

#### Available sizes

Sizes	1	2	3
Arm circumference (cm)	22-30	30-38	38-46
Reference	SRT238M1	SRT238M2	SRT238M3

The circumference of the arm in proximal 1/3 is measured

**The orthosis can be used on both the left and right shoulder.**

#### Indications

It is used in case of inflammatory and degenerative rheumatic processes, various traumatic conditions, for immobilization after humerus fracture without movement, when it is not necessary to immobilize the shoulder and elbow.

The recommendation is made by the specialist physician.

#### Description

Orthosis made of textile, provided with:

- one band under the opposite armpit, which has a protection sleeve
- 2 straps with hook and loop fastening on the arm
- 4 plastic components on the arm, in inner pockets, which produce a rigid, circular immobilization. For a better fit on the arm, the 2 plastic components from the sides can be removed (item 3 and 4)
- adjustable support strap (with "crocodile" type fastener and protection sleeve), optional.

The orthosis is a semirigid support that covers the shoulder and the arm, and can be used both as an orthopedic treatment and for recovery in the post surgery period.

Due to the fastening strap provided with 2 buckles the orthosis can be worn both on the left and right shoulder. The length can be adjusted by cutting at the level of the stitches to avoid fabric breaking, and repositioning the "crocodile" type fastener.

#### Purpose

The orthosis is a semirigid support that immobilizes the shoulder and the arm.

#### Composition

Laminated textile. The laminated textile ensemble behaves like a moisture regulator, ensuring a quick transfer of the moisture to outward.

#### Positioning

The orthosis is fixed on the shoulder and arm, after which the band is fixed and adjusted to the optimum compression under the opposite armpit.

To shorten the strip, remove the "crocodile" type hook and loop fastening from one of the ends, cut the tape to the optimum length and reposition the fastener.

The optional support strap is attached to the wrist and around the neck.

If it is necessary to adjust the position of the orthosis, the adjustment is made by fully opening the hook and loop fastener and repositioning the orthosis higher or lower on the body segment, depending on the need. **To avoid the damage of the material, do not pull the orthosis while is fixed.**



#### Cleaning

To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

#### Storage

At room temperature, in a dry space with good ventilation.

**Commercial warranty of the product: 30 days.**

**Date of production:** see packaging label.

**Validity date:** no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.


**Deadline for communication of the solution: 15 days.**

Please send notifications and complaints to the following email address: [suportclienti@triamed.ro](mailto:suportclienti@triamed.ro)

WARRANTY COUPON	
30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller name and signature

SR EN ISO 9001:2015

SR EN ISO 13485:2016

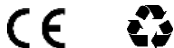

**Manufacturer SC TRIAMED SRL**

**Production** Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

**Registered office** Str. Cosminului nr.12 | Timișoara | Timiș | România

**Phone** 00 40 256 487388

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 6, Revision 0/28.07.2025