

## ANKLEFIX CONFORT

Mobile ankle support with back fastening

Ref: SRT 311

### INSTRUCTIONS FOR USE



Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests.

These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

The orthosis is used according to the recommendations of the specialized physician.

**Choosing the size** - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

#### Available sizes

Sizes	1	2	3	4	5
Ankle circumference (cm)	20-22	22-24	24-26	26-28	28-30
Reference	SRT311M1	SRT311M2	SRT311M3	SRT311M4	SRT311M5

The circumference of the ankle is measured around the malleolus

#### Indications

It is used in cricks, luxations, post fractures, after consolidated fractures, inflammatory processes, after the removal of the orthopedic plaster cast.

#### Description

The orthosis is made of textile, provided with 2 fastening straps.

The heel area is open.

Hook and loop fastener on the back side.

Height: 20 cm.

The orthosis is a support for the ankle, ensuring stability while wearing.

#### Composition

Three-layer laminated textile. The laminated textile ensemble behaves like a moisture regulator, ensuring a quick transfer of the moisture to outward.

#### Positioning

The orthosis is placed on the foot so that the straps are fastened on the back side to the optimal compression.

#### Cleaning



To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

#### Storage

At room temperature, in a dry space with good ventilation.

**The legal guarantee of conformity** according to Law no. 449/2003: **2 years.**

**Commercial warranty of the product: 30 days.**

**Date of production:** see packaging label.

**Validity date:** no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

**Deadline for communication of the solution: 15 days.**

Please send notifications and complaints to the following email address: [suportclienti@triamed.ro](mailto:suportclienti@triamed.ro)

<b>WARRANTY COUPON</b> 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller signature and stamp



**Manufacturer: TRIAMED SRL**

România/Timiș/Timișoara, Str. Calea Buziașului nr.77, Hala 1-C1/A

Tel/Fax: 00 40 256 487388

[office@triamed.ro](mailto:office@triamed.ro), [triamed@gmail.ro](mailto:triamed@gmail.ro)

[www.triamed.ro](http://www.triamed.ro)

Edition 3, Revision 2/08.11.2022