

## MALLEOFIX PLUS JUNIOR

Mobile ankle support with elastic strap

Ref: SRT 331

### INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

#### Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

#### Available sizes

Universal size

For ankle with circumference between 14 – 20 cm.

#### Indications

It is used in cricks, luxations, after consolidated fractures, inflammatory process, after the removal of the plaster cast..

#### Description

Orthosis with anatomical design, made of textile, provided with:

- 2 elastic straps crossed around the ankle for compression and fastening, which can be repositioned as needed
- 2 flexible metallic splints, one on each side
- the 2 elastic bands each have a pocket for the flexible metal splints. These elastic bands can be removed or repositioned depending on the ankle circumference..

The heel area is open.

Hook and loop fastener on the front side.

Height: 20 cm.

The orthosis is a support for the ankle joint, the 2 flexible metal splints ensuring stability and safety in use. The elastic bands crossed around the ankle over the 2 splints provide additional compression, making the orthosis an effective support.

#### Purpose

The orthosis is an effective support for the ankle joint, the 2 flexible metal splints ensuring stability and safety in use.

#### Composition

Laminated textile: PA Polyamide – 51.2%, PU Polyurethane – 43.2%, EA Elastane – 5.6%. The laminated textile ensemble behaves like a moisture regulator, ensuring a quick transfer of the moisture to outward.

#### Positioning

The orthosis is placed on the foot so that the straps are fastened in the front and the long bands are crossed under the sole and are fastened in the back to the optimal compression level.

#### Cleaning



To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

**Storage** At room temperature, in a dry space with good ventilation.

**Commercial warranty of the product: 30 days.**

**Date of production:** see packaging label.

**Validity date:** no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

**Deadline for communication of the solution: 15 days.**

Please send notifications and complaints to the following email address: [supportclienti@triamed.ro](mailto:supportclienti@triamed.ro)

<b>WARRANTY COUPON</b>	Batch and date of manufacture
30 days from the date of purchase	
Product reference	Date of purchase
The reason of complaint	Seller name and signature

SR EN ISO 9001:2015  
SR EN ISO 13485:2016



**Manufacturer SC TRIAMED SRL**

**Production** Calea Buziașului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

**Registered office** Str. Cosminului.nr.12 | Timișoara | Timiș | România

**Phone** 00 40 256 487388

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 6, Revision 0/20.04.2025