

STOMATEX DUO

Abdominal stoma support

Ref: SRT 135

INSTRUCTIONS FOR USE



Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests.

These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

Available sizes

Sizes	1	2	3	4	5	6
Waist circumference (cm)	65-75	75-85	85-95	95-105	105-115	115-125
Reference	SRT135M1	SRT135M2	SRT135M3	SRT135M4	SRT135M5	SRT135M6

The circumference of the waist is measured

Indications

It is used after abdominal surgery, by people after colostomy or urostomy.

Description

The support is made of elastic fabric.

The frontal part is provided with an adjustable opening for the colostomy bag.

Hook and loop fastener on the front side.

Width: 20 cm.

It is a traditional elastic corset that due to the elastic fabric allows normal thoraco abdominal breathing.

Composition

Elastic fabric: 50% polyester, 33% polyamide, 17% elastane, with a structure that allows high air permeability in both directions.

Positioning

The orthosis is placed from the back to the front, the colostomy bag is carefully inserted through the special, adjustable opening, and then it is fastened in the front hook and loop fastener.

Cleaning



To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage

At room temperature, in a dry space with good ventilation.

The legal guarantee of conformity according to Law no. 449/2003: **2 years.**

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: suportclienti@triamed.ro

WARRANTY COUPON 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller signature and stamp



Manufacturer: TRIAMED SRL

România/Timiș/Timișoara, Str. Calea Buziașului nr.77, Hala 1-C1/A

Tel/Fax: 00 40 256 487388

office@triamed.ro, triamed@gmail.ro

www.triamed.ro

Edition 3, Revision 2/07.11.2022