

## ABD JUNIOR 01

Support for abduction

Ref: SRT 322



### INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests.

These orthoses meet the essential requirements of Government's Decision 54/2009 and have registration Certificates from the Ministry of Health.

These products satisfy the European Medical Device Directive 93/42/CEE (RD.1591/2009).

The orthosis is used according to the recommendations of the specialized physician. The product must be adjusted by the orthopaedic technician!

**Choosing the size** - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

#### Available sizes

Sizes	1	2	3	4	5
Weight of the child (kg)	2-3	3-5	5-7	7-9	9-11
Reference	SRT322M1	SRT322M2	SRT322M3	SRT322M4	SRT322M5

#### Indications

It is used in the case of congenital hip dysplasia.

#### Description

It is made of textile, provided with a rigid pad for keeping legs in the abduction.

Hook and loop front closure.

#### Composition

Textile fabric – 100% cotton.

Rigid pad – 100% polyethylene foam.

#### Positioning

Position the support for abduction from back to front so that the thighs are in abduction.



#### Cleaning

To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

#### Storage

At room temperature, in a dry space with good ventilation.

**The legal guarantee of conformity** according to Law no. 449/2003: **2 years**.

**Commercial warranty of the product: 30 days.**

**Date of production:** see packaging label.

**Validity date:** no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

**Deadline for communication of the solution: 15 days.**

Please send notifications and complaints to the following email address: [suportclienti@triamed.ro](mailto:suportclienti@triamed.ro)

<b>WARRANTY COUPON</b> 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller signature and stamp



**Manufacturer: TRIAMED SRL**

România/Timiș/Timișoara, Str. Calea Buziașului nr.77, Hala 1-C1/A

Tel/Fax: 00 40 256 487388, Tel: 00 40 765 240037

[office@triamed.ro](mailto:office@triamed.ro), [triamed@gmail.com](mailto:triamed@gmail.com)

[www.triamed.ro](http://www.triamed.ro)