


THORATEX 04

Rigid thoracic support

Ref. SRT 140

INDICATIONS

It is used to correct vicious positions of the thoracic spine, or in case of broken collarbone.

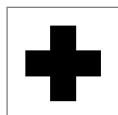

MEDICAL



SPORT



DAILY

DESCRIPTION

Support with anatomical design, made of 3 rigid textile fixing straps:

- 2 straps that go around the shoulders correcting the body posture by traction
- one back strap, used for the final adjustment.

The 3 straps are adjustable. To shorten the straps, remove the "crocodile" type fastening from the extremities, cut the strap to the optimum length on the prefigured transverse marks and reposition the hook and loop. The two straps that surround the shoulders are provided with sleeves for underarm protection.

The orthosis is made of a soft, rigid fabric that does not harm while wearing. The big surface of the hook and loop fastening allows a good adjustment and strong traction, helping the clavicle to return to its anatomical shape after fracture.

For a better result it is recommended to readjust during wearing.

Hook and loop fastener on the front side.

COMPOSITION

The rigid textile made of polyamide fiber has a high resistance to chemical and biological agents, resists abrasion and does not absorb water.

AVAILABLE COLOURS

Black



AVAILABLE SIZES

Universal size

For thorax circumference between 75 - 135 cm.

